

Taffix™ - user instruction

Read this leaflet carefully before using the Taffix™

Intended use

- The Nasus Nasal Spray Device (Taffix™) is intended for use to remove particles from the air for medical purposes.
- The Nasus Nasal Spray Device (Taffix™) is intended for Over-The-Counter Use.

Indication for use

- **The Taffix™ (Powder) Spray Device is indicated for use as a protective mechanical barrier against viruses within the nasal cavity.**

What is Taffix™?

- Taffix™ is used to block viruses within the nasal cavity, viruses that could penetrate the respiratory system and cause infection.
- To protect against airborne viruses, Taffix™ should be used prior to exposure to a high risk environment where viruses may be airborne, such as public place (e.g., supermarkets, pharmacies, school, sport club, flight, etc.) or encounters with anyone outside of the isolated members of the household.
- Taffix™ is aimed for intermittent use: use it when you are in crowded areas. There is no need to use it at home or where you are not exposed to other people who might infect you.

How does Taffix™ work?

- Taffix™ is a powder nasal spray, the powder turns to a gel in the nose and creates an active barrier that shields, protects and deactivates airborne viruses found often in circulated air in open spaces, planes, trains, large shopping centers and public transportation.
- If concurrent use of another nasal spray is required, Taffix™ should be re-applied afterwards so that the barrier created is not disturbed.
- Taffix™ can be used in addition to other treatments, as part of combination therapy strategy in the fight against viruses.
- Taffix™ may be used in addition to donning a mask and wearing gloves.
- Taffix™ is not a drug and does not cause drowsiness.

Who can use Taffix™?

Taffix™ is suitable for adults, athletes, and children from 12 years old.

What are the contraindications?

- Do not use this product if you are sensitive to Cellulose, Citric Acid, Sodium Citrate, Benzalkonium Chloride or Menthol.
- Consult a physician before using Taffix™ in children below age 12 or pregnant or breast-feeding women.

What is the chemical composition?

Hypromellose (HPMC) (89.9%), Citric Acid (6%), Sodium Citrate (4%), Benzalkonium Chloride (0.1%), Menthol (<0.1%).

⚠️ What are the cautions?

- Do not use after expiration date or 6 months after first opening - whichever comes first.
- May be used up to 3 times a day. If you are using Taffix™ 3 times a day every single day-with no days off-you should stop after 30 days.
- Keep out of the sight and reach of children.


- Avoid contact with open wounds.
- Avoid contact with eyes. If powder gets into your eye, rinse with water.
- Do not rinse the bottle with water or other liquid as this may block the bottle.
- Do not use if tamper evident seal is broken.
- Always close the cap.
- Very few cases of allergic reactions to this product have been reported.
- Most reports of side effects have been mild - blocked nose, runny nose, sneezing, sore throat. If you experience reactions more severe than these - then the use of the product should be discontinued and if symptoms persist, consult your physician.
- Remember to squeeze the bottle gently to avoid excess powder emittance.

How to store Taffix™?

- Store at room temperature or between 5°C ~ 30°C (41°F ~ 86°F).
- Do not refrigerate or freeze.

How to use Taffix™?

- To defend against airborne viruses, Taffix™ may be applied up to 3 times per day to maintain an effective barrier against viruses.
- **Taffix™ bottle should be used for one person only to avoid contamination - Mark your bottle and avoid sharing it with other people to prevent potential transmission of viruses or germs.**
- The bottle is intentionally not filled completely since the air within the bottle is needed for optimal spraying the powder to the nose.

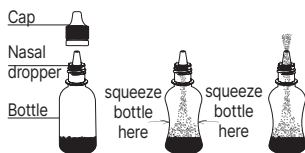

- The Taffix™ bottle contains 1g of gentle powder
- Taffix™ bottle is sufficient for up to 200 sprays and should be applied up to 3 times per day.
- Once applied, Taffix™ becomes effective after a minute and is effective for approximately 5 hours.

How to prepare a new Taffix™ for use?

- Take your Taffix™ Nasal Spray out of the package and open the cap



Shake Taffix™ bottle



Test spray into the air twice

- The Taffix™ is now ready for use

How to use the Taffix™?

Every time you take Taffix™ follow the instructions below



Blow your nose



Shake the bottle



Apply 1 to 2 sprays into each nostril



Wipe the bottle

- During first use, you may experience a tingling or burning sensation in the nose especially if the nasal tissue is sensitive or inflamed.
- You may also feel some form of irritation, especially if your nasal membranes are already inflamed or irritated. This will subside quickly.
- Some individuals can feel a slight sensation or blocked nose, this is normal and is the feel of the barrier created by Taffix™. If you experience continuous discomfort you can wash your nose with warm water until relief. If you experience serious discomfort after application of Taffix™ consult your physician.

Taffix™ can be bought in several configurations:

REF 1841600 Taffix™ - 1 bottle of 1g powder

REF 1841602 Taffix™ Travel - 1 bottle of 0.7g powder

REF 1841603 Taffix™ Family - 4 bottles of Taffix™ (REF 1841600)

Manufacturer: Nasus Pharma Ltd.
Harakevet 29 Tel Aviv, 6618306, Israel

Distributor: Kiara MedTech (Pty) Ltd
Address: 72 Steel Road, Spartan, Kempton Park, 1619, South Africa
Tel: +27 10 3290600
Email: info@kiarahealth.com
Website: www.kiarahealth.com

Symbols Glossary:

	Legal Manufacturer
	CE Mark, indicating that the device complies with Council Directive 93/42/EC (MDD)
	Caution, consult accompanying documents
	Consult instructions for use
	Batch Number
	Catalogue number
	Do not use if package is damaged
	Minimum and maximum storage temperatures
	Keep away from sunlight
	Expiry date (In format: YYYY-MM)
	Keep dry